Asado first aid kit

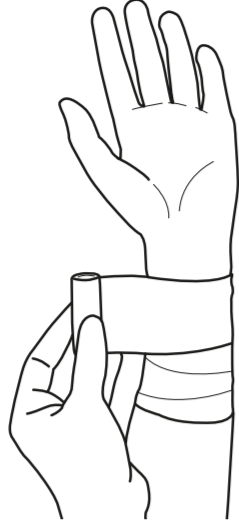
Danyang Sunmed Healthcare Corporation No. 3, Bei Er Huan, 712380, Danyang, Zhejiang, China


Shanghai International Holding Corp. GmbH (Shanghai) Effelstraße 80 20317 Hamburg, Germany

Lange Kleiweg 6-28, 2288 GK Rijswijk, the Netherlands

1 X ELASTIC GAUZE BANDAGE 4 M X 6 cm

4 X ALCOHOL PAD 6 X 3 CM

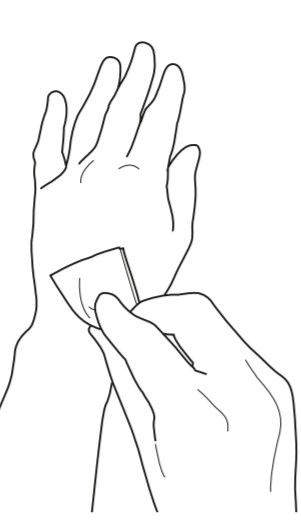21 X ADHESIVE BANDAGE 50 X 100 MM 18 X 70 MM 10 X 40 MM

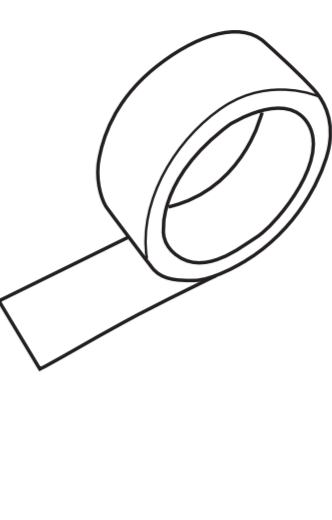
1X TRIANGULAR BANDAGE 96CM X 96CM X 136CM

4 X ANTISEPTIC CLEANSING WIPE 6 X 3 CM

1 X NON-WOVEN ADHESIVE TAPE 1,25 CM X 4 M

1 X SCISSORS

PBT BANDAGE
Intended use: Used to keep compresses or other medical dressings in place or to fixate compresses or other medical dressings.
Instruction for use: Wrap the bandage around the compress or dressing.
Do not apply excessive pressure. Place the end under the lower layers or use the adhesive tape to fixate the bandage.
Warning! For single use only. Non-sterile. Store in a cool and dry place. Do not apply directly onto the wound or injured skin. Release the bandage to reduce the pressure in case of discomfort. Use before the expiry date.

ALCOHOL PAD
Intended use: Used to clean the skin and the area around minor wounds.
Instruction for use: Clean the wound from the centre outwards to avoid getting bacteria into the wound. Throw the cleansing wipe away after use.
Warning! For single use only. For external use only. Do not use if the package is damaged. Avoid contact with the eyes. Store in a cool and dry place. Use before the expiry date.

ADHESIVE BANDAGE
Intended use: To be used on minor wounds or cuts.
Instruction for use: Clean the wound and the surrounding skin. Position the wound pad at the wound after the skin is dry. Replace the plaster strip every 24 hours or when it is wet.
Warning! For single use only. Store in a cool and dry place. Store below 25 °C. Do not use if the package is damaged. Use before the expiry date.

TRIANGULAR BANDAGE
This triangular bandage is a comfortable sling made of soft materials for additional comfort.
Instruction for use: This provides shoulder and arm support to relieve pain and immobilize the arm. Use before the expiry date.

ANTISEPTIC CLEANSING WIPE
Intended use: Used to clean, dry or protect the wound and the surrounding skin.
Instruction for use: Wipe the compress into the area that needs to be cleaned or dried. Or place the compress onto the wound that needs to be protected.
Warning! For single use only. Do not use if the package is damaged. Store in a cool and dry place. Use before the expiry date.

NON WOVEN ADHESIVE TAPE
Intended use: Used to fixate compresses or other medical dressings.
Instruction for use: Use the adhesive tape to keep the compress or other medical dressing in place. Do not apply the adhesive tape too tight.
Warning! For external use only. Store in a cool and dry place. Store below 25 °C. Keep away from children. Do not apply directly onto the wound or injured skin. Remove the tape if the skin becomes irritated and consult a doctor.

SCISSORS
Intended use: Used to cut bandages, gauze, dressings or other small first aid devices.
Warning! Keep away from children.

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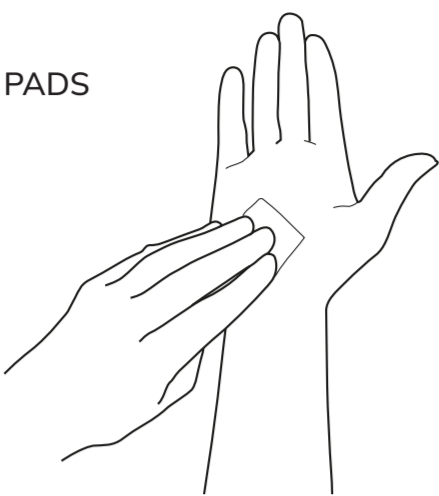
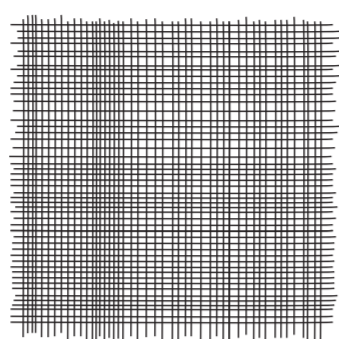
1 PAIR
DISPOSABLE GLOVES
24 X 15 CM

UK

STING RELIEF PREP PADS
Intended use: Help to prevent infection in minor scrapes and offer temporary relief of itching from insect bites. Instruction for use: Apply topically as needed to cleanse the intended area. Warning! For single use only. Do not use if the package is damaged. Store in a cool and dry place. Use before the expiry date.

GAUZE PAD
Sterile gauze pad, for first aid. Intended use: Used to protect wounds and to absorb blood or other fluids. Warning! For single use only. Store in a cool and dry place. Use before the expiry date.

DISPOSABLE GLOVES
Intended use: Disposable gloves help to protect against the transmission of disease and illness. Note that the responder must still wash his or her hands after the gloves are removed to eliminate the risk of disease transmission. Warning! For single use only. Do not use if the package is damaged. Store in a cool and dry place. Use before the expiry date.

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Danyang Sunmed Healthcare Corporation No. 3, Bei Er Huan, 212300, Danyang Jiangsu, China
Shanghai International Holding Corp. GmbH (Europe) Effelstraße 80 20517 Hamburg, Germany